



SoundingBoard App

Creating a New Communication Board


1. Select/Open the SoundingBoard app

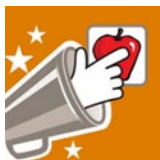


2. Select Add a Board in the top right corner

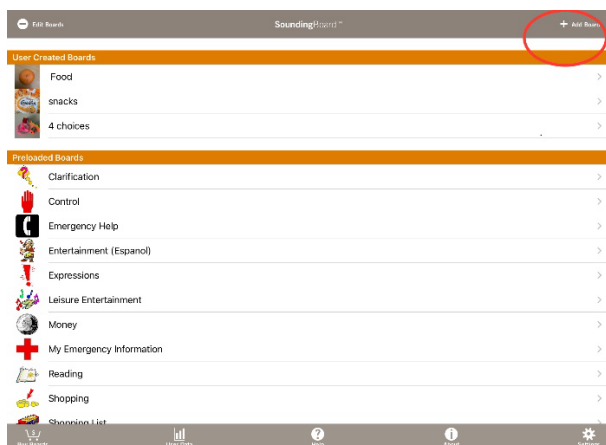


3. Title the communication board

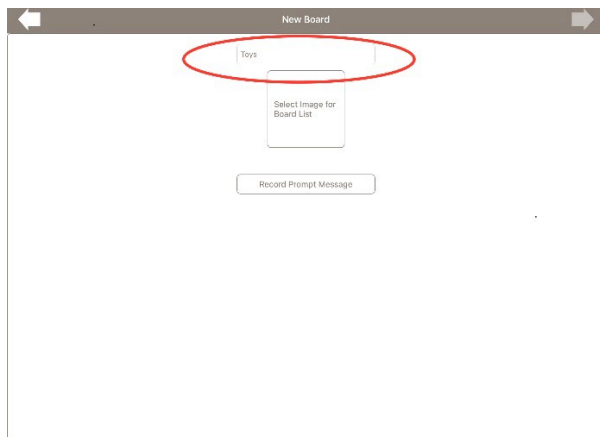

4. Add a photo or select a symbol from the library

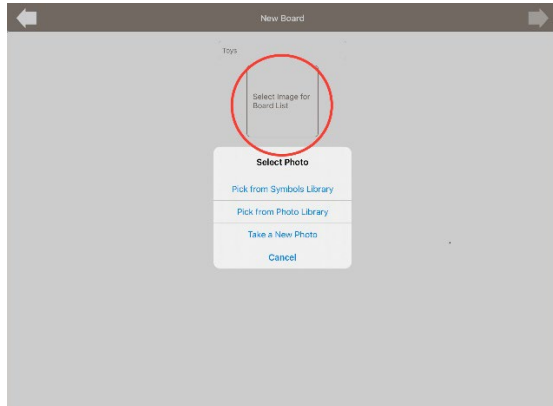

5. Select the arrow in the top right corner to save your changes/edits

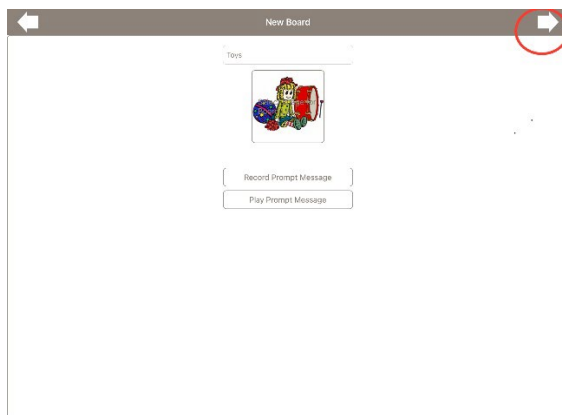

6. Once you are in the blank communication board select add image to begin adding vocabulary and customizing your new communication board. SoundingBoard app will allow 1-20 messages to be programmed. Contact your SLP for instructions about PCS vs Photo and how many messages per board is best for your student.




7. Step 1: Add your image (Photo from camera, library or by selecting a symbol)

Step 2: Title your vocabulary item


Step 3: Record a message (e.g. Apple, I want an apple etc)

Step 4: Select the arrow in the top right corner to save the changes and edits made

***** if all steps are not completed the app will not save the changes made**

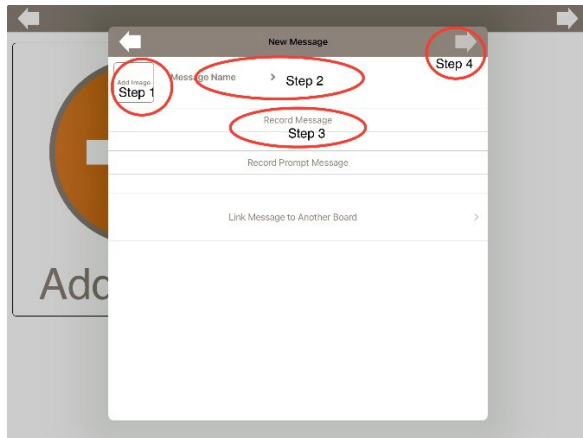


8. Turn edit mode off by Selecting Edit board. You will notice the red circles will disappear

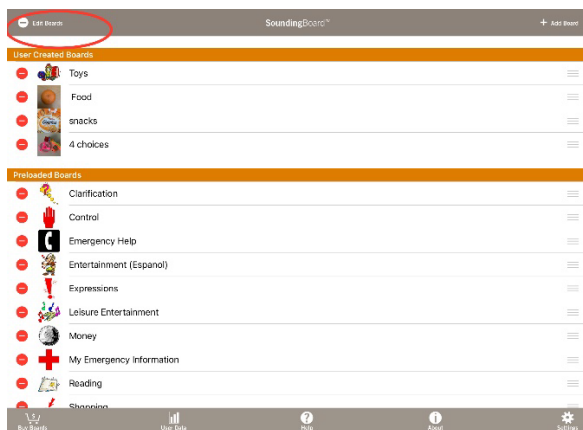


Deleting a Button

1. Select Edit Boards in the top left corner to make changes to a communication board

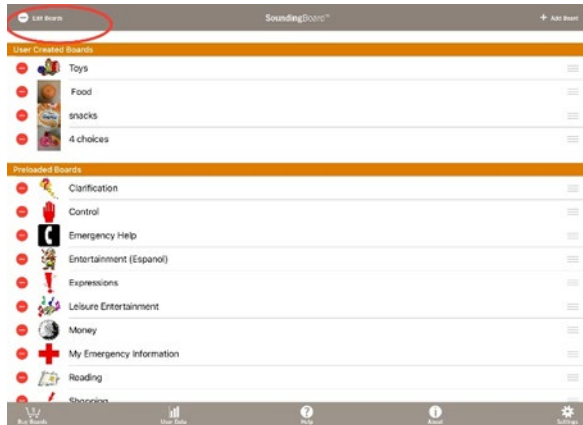

2. Select the communication board you would like to edit (e.g. Toys)

3. Once the communication board has been selected, tap the photo you would like to edit. You will be provided an option to delete or edit. Once you have deleted or edited the image select the top right arrow to save your edits/changes.

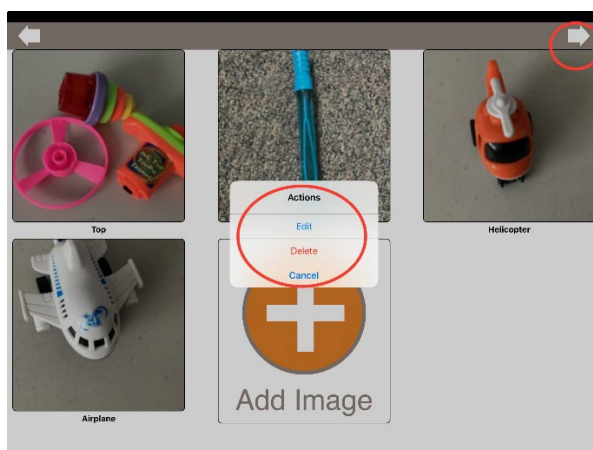

4. On the main page slide Edit Boards off



Watch the SoundingBoard “How to” Video:

