


TD Snap App “How to” Manual




Adding Vocabulary


1. Select the Settings symbol.

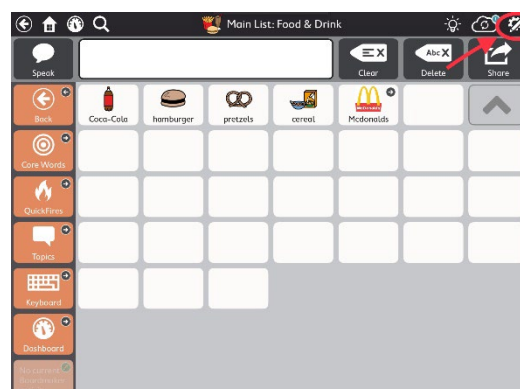


2. Select the “+” to add a button.

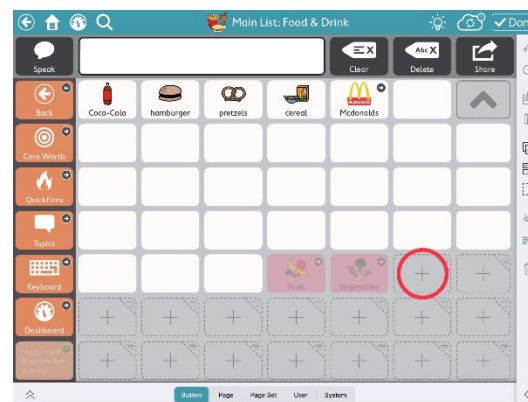


3. Type in your item

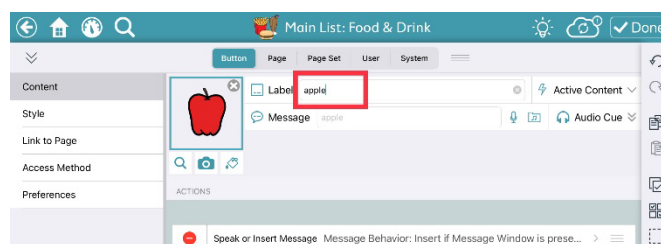


4. To change the symbol, select the current symbol and choose a different one

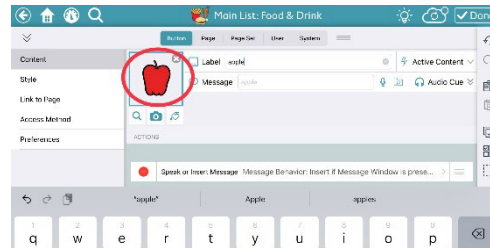

5. Select “Done” to save

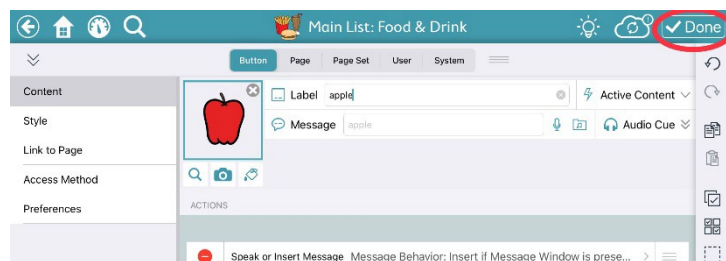

6. To take a photo rather than selecting an image, press the camera icon

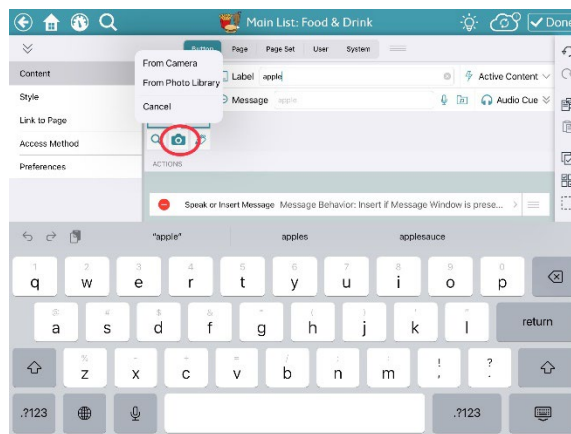

7. Choose the photo from the camera roll or take a photo

8. Remember to hit “Done” to save your edit

Watch the “How to add a button” video:



Moving a button


1. Select the Settings button.
2. Press and hold the button you want to move until it becomes translucent.
3. Move the button and wait until the other button you are replacing moves.
4. Let go of the button.
5. Select “Done” to save your edit

Watch the “How to move a button” video:



Adding a Folder

1. Select the Settings button. Select the shaded tab at the corner of a blank button to add a folder

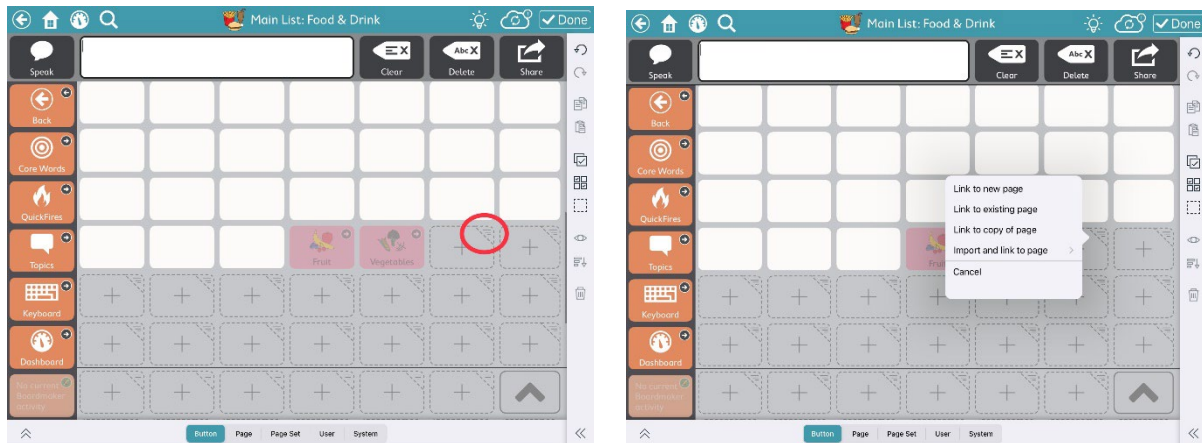

Link to New Page: This is an empty folder without any pre-loaded vocabulary.

- a. Select “Link to New Page”.
- b. Add the name of your folder.
- c. Change the symbol if you want something different
- d. Select “Create”.
- e. Select “Done” to get out of Edit mode.
- f. Start manually adding vocabulary to your folder.

Link to Existing Page: When creating a new folder you have the option to link it to a folder that already exists.


- a. **Select “Link to Existing Folder”.**
- b. **In the Search Bar, enter the folder you want to link to and select it.
A blue checkmark will appear on the left side of the menu.**
- c. **Select “Create”.**
- d. **Select “Done” to get out of Edit mode.**

Watch the “How to add a folder” video:



Search Function

1. **Select the magnifying glass symbol**

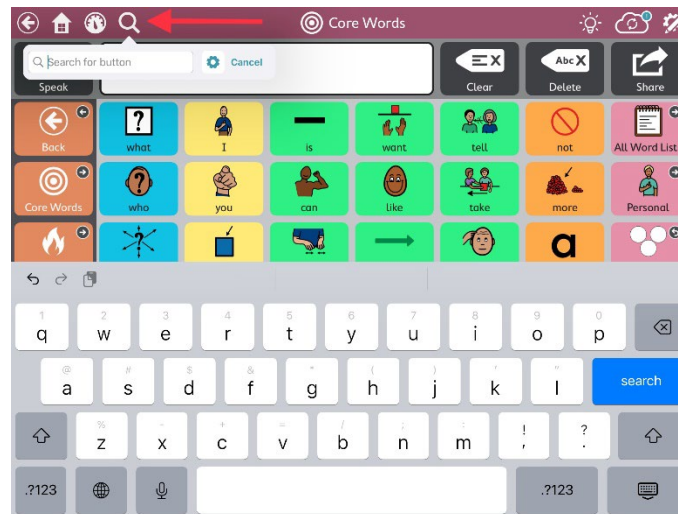


2. **Type the name of the item you are looking for in the window (e.g. bubbles)**
3. **Click on the desired item and follow the navigation to its location.**

Watch the “How to search vocabulary” video:




Backing up your student's device

1. Click on the cloud icon in the top right corner of the screen.

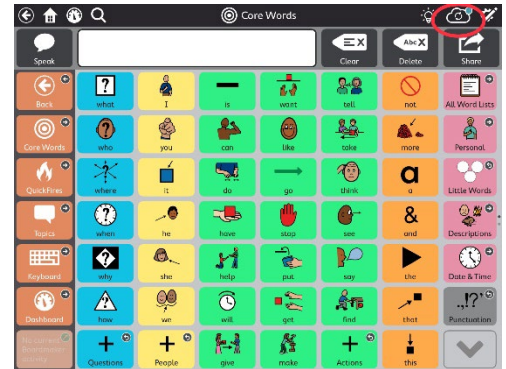


2. Select "Sync Page Set".

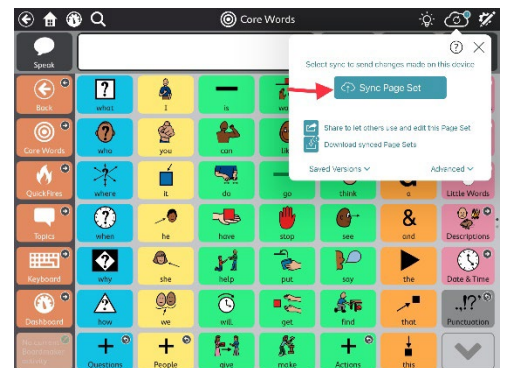

3. Wait until the page set has been synced. You will get a notification that it is done.

4. Click the "X" to exit out of the screen

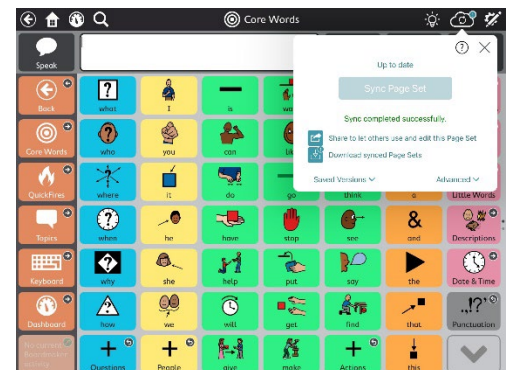

Watch the "How to back-up the device" video:

